

# USER'S GUIDE

## JORDAN H



**RESI**<sup>®</sup>  
[www.resi.cz](http://www.resi.cz)

Manufacturer: RESI Třeboň spol. s r.o., Novohradská 1153, 379 01 Třeboň

Medical device options:

**JORDAN H, H2e, H23e, H3e, H URL**

**Medical device dedicated purpose:** JORDAN H series tables are intended for gynaecological and urological ambulances for common operations in medical facilities, as well as examination couches during continuous supervision by medical personnel. Chairs with URL identification are intended for urodynamic examinations, measurement, and for video-urodynamics using the C mobile arm.

**Important notice - the JORDAN table is not suitable for explosive environment. The device is not suitable for usage with the presence of inflammable anaesthetics and air mixtures, or inflammable anaesthetics and oxygen mixtures, or monoxide nitrogen. Usage of explosive anaesthetics is dangerous.**

**Risk class:** In terms of § 6, par. 1, letter a) of Act No. 268/2014 Coll., on medical devices, and in terms of Act No. 634/2004 Coll., on administrative fees, as amended, the medical device set forth above is classified in compliance with Attachment No. 9 of the Government Regulation No. 54/2014 Coll. in the Risk Class I.

**Medical device sterility:** The medical device indicated above is put in the market as a NON-STERILE device.

**Measuring features:** The medical device indicated above is put in the market as a NON-MEASURING device.

Specifications:	
Table weight	70 - 110 kg
Dimensions of JORDAN H, H2	127 x 65 cm
Dimensions of JORDAN H23	195 x 70 cm
Dimensions of JORDAN H3	200 x 65 cm
Maximum couch load	180 kg

**Couch installation steps:**

1. Remove the packaging materials.




2. Adapt to the floor unevenness using the levelling screw (JORDAN H3)



3. Insert fibular rests and tighten them in proper position.



4. Insert elbow rests to the structure (JORADN H, H2, H23, H URL)

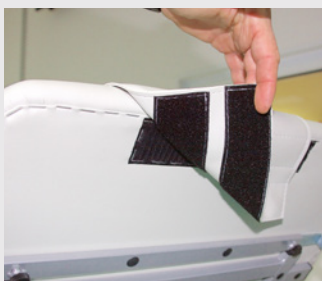

5. Attach the semi-cylinder using Velcro elements



Manual control



Pedal control



The dish is easy to remove

## Couch operation:

JORDAN H



JORDAN H2



JORDAN H23




JORDAN URL



JORDAN H3



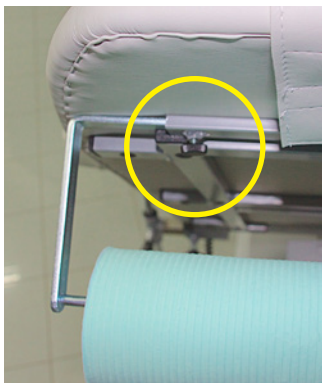

Release the holder slider by loosening the fly nut, and adjust the paper size.



By actuating the pedal, eject castors and handle the couch. By releasing the pedal, fix the couch again.  
(JORDAN H3)



After removing the footrests from the chair, the foot segment can be inserted (see figure) to convert it to the couch. In this way, you can even obtain the Trendelenburg position (JORDAN H23)

## Safety

### Safety instructions:

- Do not allow children to use and operate the device.
- Do not overload above the maximum limit of 180 kg.
- Do not sit on the head section raiser with the maximum load of 60 kg, in order to prevent from injury or couch permanent damage.

### Potential failures and troubleshooting:

- The head section raiser does not keep its position even under a small load: you can set the spring pressure and the spring resistance force by rotating the gas piston body clockwise.
- When the failure persists, contact the service.

## Special conditions for storage and maintenance:

Good quality coaches require proper care to prevent from preliminary devaluation.

Keep the coaches away from excessive moisture and heat.

Protect the upholstery against shocks from sharp edges of other furniture, and against sunshine.

Clean the dust away with wet cloth.

You can disinfect the couch parts with INCIDIN LIQUID disinfection agent, which is delivered with the couch including a sprayer. You can also order these disinfection agents later from RESI Třeboň spol. s r.o. company.

You can remove contamination from all parts of the couch using detergents, such as Corona, Universal, etc., with sponge or cloth. Do not use any organic solvents for the cleaning (benzol, trichlorethylene, acetone, etc.).


In order to treat chromed parts, use Neoxyd, Silichrom, etc.

## Alerts and other important warnings:

### Guarantee, guarantee conditions:

Providers (of health care) are bound to assure the following 2-year guarantee conditions:

- Install, use, and maintain medical devices for the intended purpose, and in compliance with operating rules (user's guide) of the manufacturer, and regulations for occupational and health protection.
- Keep the operating documentation of the medical device.
- The manufacturer sets the obligation to perform a safety/technical inspection at least every 12 months, provided by the manufacturer.

 The couch lifetime is max. 10 years since the delivery date. After this period, complete audit of the couch must be performed by the manufacturer. The couch may not be used within the health service without the audit!! The manufacturer performs the audit based on the order by the medical device user. Based on the complete audit, the couch lifetime is extended by 5 years.

### Caution!

If the instructions included in this user's guide are not observed, the manufacturer is not liable for damages, if any, resulting from incompetent operation or treatment.

User's guide version number: 1.1

User's guide publication date: 1. 4. 2015



**RESI**<sup>®</sup>  
[www.resi.cz](http://www.resi.cz)

RESI Třeboň spol. s r. o.

Novohradská 1153, 379 01 Třeboň, Czech Republic

Tel.: +420 384 724 292, Fax: +420 384 724 293