

Image 3



exclusive design to meet all your needs

Image 3 exclusive design to meet all your needs



Image 3 Standard



Image 3 Washable


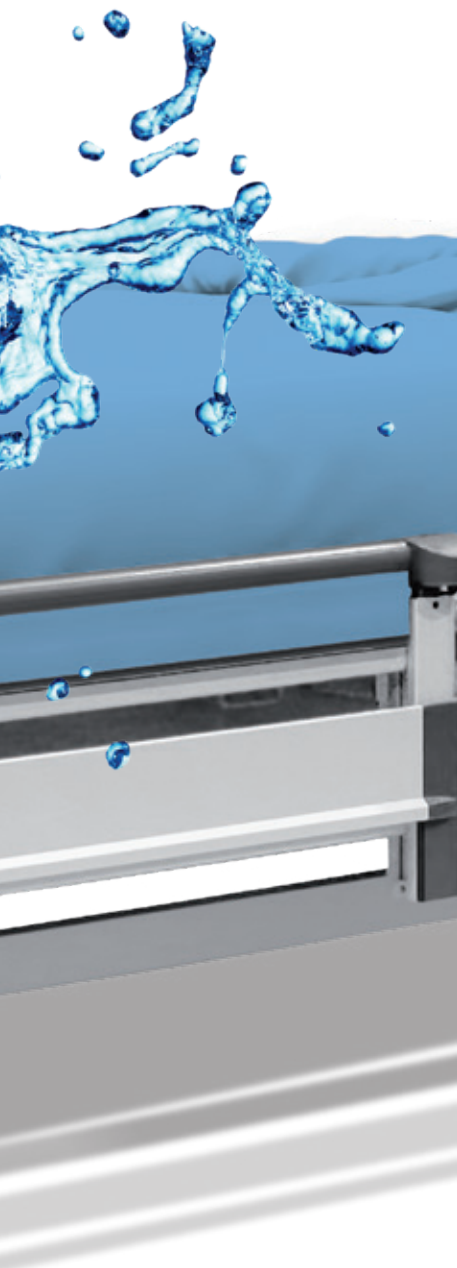


Image 3 Bariatric

Benefits of Image 3



extra
low position
28_{cm}

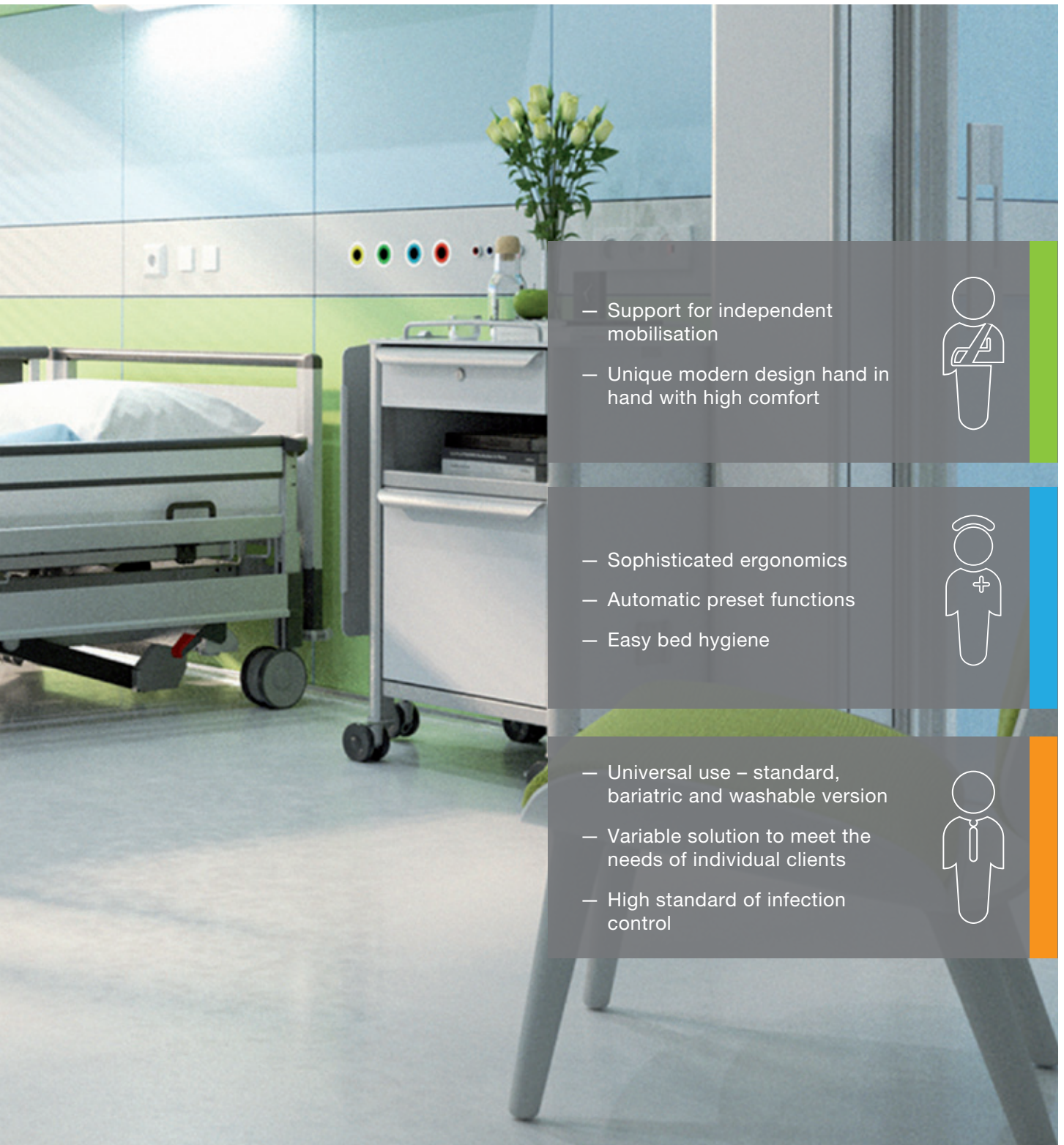

- Support for independent mobilisation
- Unique modern design hand in hand with high comfort



Patient

- Sophisticated ergonomics
- Automatic preset functions
- Easy bed hygiene



Staff

- Universal use – standard, bariatric and washable version
- Variable solution to meet the needs of individual clients
- High standard of infection control



Management

Meeting everyone's needs



Image 3 WASHABLE

The bed is available in a version suitable for cleaning and disinfection in automatic hospital washing machines.

WASHABLE
FRIENDLY

- The bed construction undergoes a special technological modification that increases its service life and durability.

Image 3 STANDARD

The modern design of the Image 3 bed will find its universal use in hospitals.

- By choosing the version, design and accessories, the bed can be adapted for a specific purpose.
 - The attractive design of the bed helps to create a comfortable and pleasant environment in health facilities.





Image 3 Bariatric

Bariatric version of Image 3 meets the growing demand for beds with higher load capacity.

- The bed has a wider mattress platform.
- The safe working load is increased to 320 kg.

WIDER BED

HIGHER LOAD

Variable design

HOSPITAL
DESIGN



EXCLUSIVE
DESIGN



UNIVERSAL
DESIGN



The design of the bed is timeless and suitable for a broad spectrum of needs in hospital care. Image 3 Standard, Washable and Bariatric fully respect the highest ergonomic principles in healthcare.



Due to its classic concept, the hospital design suits standard healthcare practice and in addition offers a version specially adapted for cleaning in automatic hospital washing machines.



The exclusive design is enhanced by a combination of aluminium elements and wooden decors. The elegant look has a comforting effect and creates a pleasant environment for both the patients and staff.



The simple design offers a compromise of the bed's unique practical features in an economic version.

Patient safety

Safety is a key parameter for the Image 3. All the bed's functions are designed so that the patient can not just feel, but really be as safe as possible.


Ergoframe®

The Ergoframe® system extends the mattress platform in the pelvic area and reduces the pressure load on the tissues in this region when a patient is positioned in the cardiac chair position.



Functions lock


The electrical functions of the bed can be simply locked.

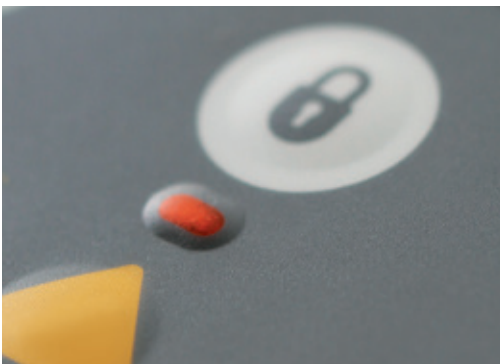

BRAKE Alarm

Brake alarm

The bed automatically emits a audible signal when the bed is left unbraked while the main power cable is connected.

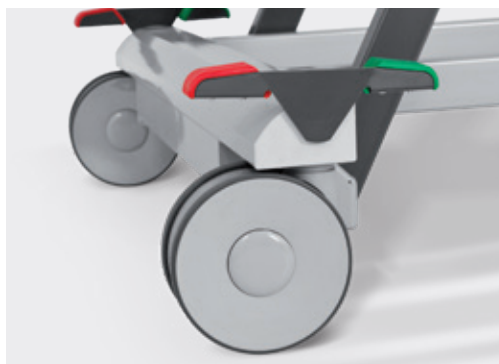

GO Button

Pressing the 'GO' button activates the control panel for a period of up to 3 minutes from the last push. The controls are therefore locked during periods of inactivity and cannot be accidentally activated.



SafeFree[®] siderails

4 different adjustable heights of siderails. No required additional space beside the bed folding. Covered full length of bed sides.



SafeSense

The SafeSense bed exit alarm warns the staff when the patient leaves the bed.



3-STOP strategy

EXAMINATION
POSITION



MOBILISATION
POSITION



EXTRA LOW
POSITION





Examination position

- The highest bed position provides the staff with ideal ergonomic conditions for work.
- Comfortable nursing procedures and care.
- The position provides complete access to the bed for cleaning and disinfecting.

80
cm



Mobilisation position

- Preset functions on the control panel for simple mobilisation.
- In combination with the siderail handle, the position simplifies and supports the mobilisation of the patient.
- The ideal setting for cleaning in an automatic hospital washing machine.

40
cm



Extra low position


- Separate button as a preset function on the control panel.
- Safe position for sleeping.

28
cm



Practical details

- 1 **5th castor**
facilitates comfortable handling in all spaces.
- 2 **Bed extension**
The mattress platform width offers great comfort to all patients. Thanks to the built-in bed extension even taller patients will feel very comfortable.
- 3 **Night light**
for the patient's safe orientation.
- 4 **3D corner bumpers**
protect the bed in a horizontal and vertical position.
- 5 **Mattress platform width**
optional mattress platform width 86 or 90 cm with removable plastic parts.
- 6 **Linen shelf**
may also be used to store the supervisor panel.
- 7 **Mechanical CPR**
for emergency use.

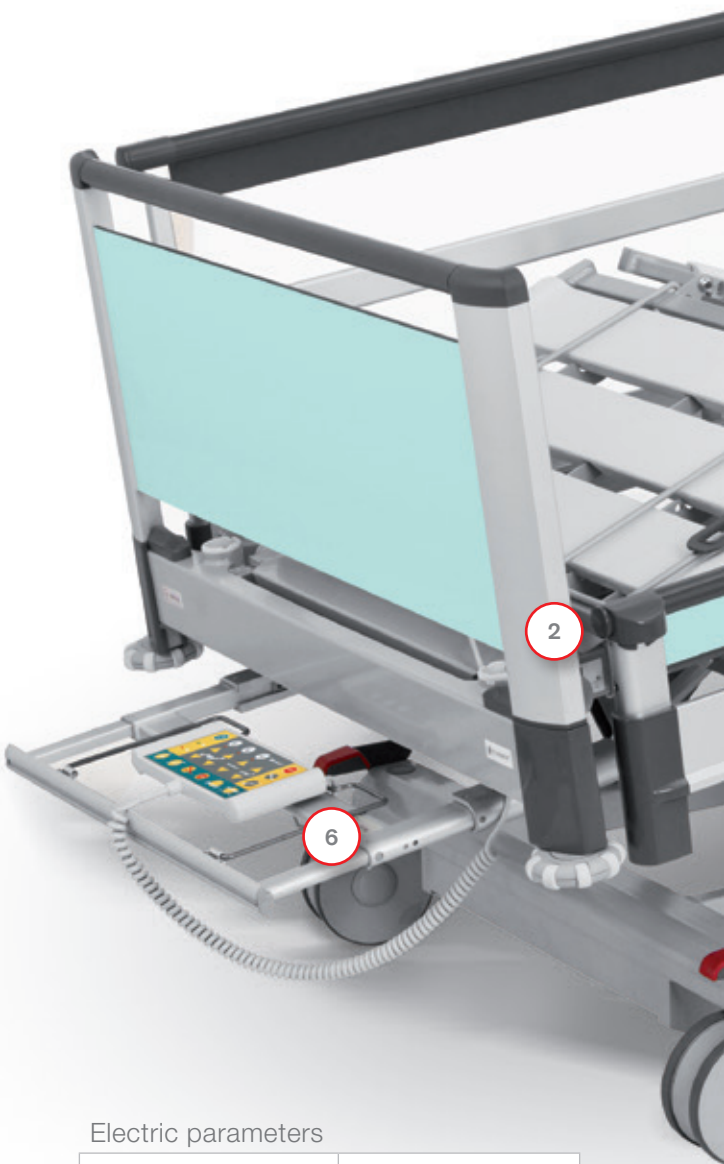

Electric parameters

Power supply	230 V/±10 %
Maximum input	max 1.6 A; 370 VA
Protection against water and dust	IPX4 (IPX6 for Image 3 Washable)

Certificates

LINET is certified according to European standards and the quality system: ČSN EN 9001:2000 ČSN EN 13485:2003
The products are manufactured according to European standards for healthcare technology and hospital beds: ČSN 60601-1, ČSN 60601-2, ČSN 60601-4, ČSN 60601-2-52, ČSN EN ISO 14941 Image 3 Washable: AK-BWA:2009

Color concept





Technical parameters	Image 3 (Standard, Washable)	Image 3 (Bariatric)
External dimensions	221 × 104 cm (split siderails) / 220 × 99.8 cm* (single collapsible siderails)	220 × 110 cm*
Mattress platform dimensions	200 × 86 / 90 cm	200 × 100 cm
Clearance	13 / 15.5 cm	13 / 15.5 cm
Mattress platform height adjustment	28 – 80 cm	28 – 80 cm
Bed extension	to 30.5 cm	to 30.5 cm
Maximum mattress height	18 cm (split siderails); 17 cm (single collapsible siderails)	26 cm including the extender
Safe working load	265 kg (Standard) / 250 kg (Washable)	320 kg
Bed weight	160 kg *, **	160 kg *, **
Backrest tilt	0 – 70°	0 – 70°
Thighrest tilt	0 – 34°	0 – 34°
TR/ATR tilt	0 – 14°	0 – 14°

* by configuration; ** by accessories

Image 3

Patient

- Support for independent mobilisation
- Unique modern design hand in hand with high comfort



- Sophisticated ergonomics
- Automatic preset functions
- Easy bed hygiene



Staff

Management

- Universal use – standard, bariatric and washable version
- Variable solution to meet clients' needs
- High standard of infection control



Members of LINET Group

LINET spol. s r. o.

Želečnice 5 | 274 01 Slaný | Czech Republic

tel.: +420 312 576 400 | fax: +420 312 522 668 | e-mail: info@linet.com | www.linet.com

